



WITH IMPLANT
SYSTEM
EQUIVALENT
REFERENCE
NUMBERS

DESS

CLINICAL SCREW RECOMMENDED TORQUE

INS-03-ENG-08

21-10-2016



ENGLISH (INTERNATIONAL)

CONNECTION	PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE [Ncm]	ORIGINAL EQUIVALENT REF. N°
EXT.HEX.UNIVERSAL (Brånemark® System)	NP	19.027	35	Nobel Biocare® 31168
	NP	19.227	35	Nobel Biocare® 29282
	NP	19.327	35	-
	NP	19.101	35	-
	RP	19.028	35	Nobel Biocare® 29290
	RP	19.228	35	Nobel Biocare® 29283
	RP	19.328	35	-
	WP	19.029	35	Nobel Biocare® 31169
	WP	19.229	35	Nobel Biocare® 29284
	WP	19.329	35	-
	NP on implant for Multi-unit®	19.501	20	Nobel Biocare® 29194
	RP on implant for Multi-unit®	19.502	20	Nobel Biocare® 29195
	TRI – LOBE (Nobel Replace® Select™)	NP	19.004	35
NP		19.204	35	Nobel Biocare® 36818
NP		19.304	35	-
RP/WP		19.005	35	Nobel Biocare® 29293
RP/WP		19.205	35	Nobel Biocare® 29475
RP/WP		19.305	35	-
NP on implant for Multi-unit®		19.504	20	Nobel Biocare® 29242
RP on implant for Multi-unit®		19.505	20	Nobel Biocare® 29243
ACTIVE HEX (Nobel Active®)	NP	19.027	35	Nobel Biocare® 31168
	NP	19.227	35	Nobel Biocare® 29282
	NP	19.101	35	-
	NP	19.327	35	-
	RP	19.030	35	Nobel Biocare® 28815
	RP	19.230	35	Nobel Biocare® 36918
	RP	19.328	35	-
	NP on implant for Multi-unit®	19.507	20	Nobel Biocare® 36892
	RP on implant for Multi-unit®	19.508	20	Nobel Biocare® 36893
	OCTAGON (Straumann® Soft Tissue Level®)	RN/WN on implant	19.026	35
RN/WN on implant		19.102	35	STRAUMANN® AG 049.181
RN/WN on implant		19.328	35	-
RN/WN on synOcta® abutment		19.009	15	STRAUMANN® AG 048.350
RN/WN on synOcta® abutment		19.103	15	-
CONICAL BL (Straumann Bone Level®)	NC on implant	19.032	35	STRAUMANN® AG 025.2908
	RC on implant	19.033	35	STRAUMANN® AG 025.4908
	NC/RC on implant	19.037	35	STRAUMANN® AG 025.4900
	NC/RC on implant	19.104	35	STRAUMANN® AG 025.4906
	NC/RC on implant	19.311	20	-
	NC/RC on implant for Multi-unit®	19.544	20	-
	NC/RC on Multi-base® abutment	19.041	15	STRAUMANN® AG 025.0900-04
	NC/RC on Multi-base® abutment	19.105	15	-
INTERNAL HEX CONIC (ASTRA TECH™)	RP on implant	19.016	20	DENTSPLY® 24449
	WP on implant	19.017	25	DENTSPLY® 24209
	UniAbutment® 20° / 45°	19.018	15	DENTSPLY® 22435
INTERNAL HEX FD (XIVE®)	NP/RP/WP	19.024	25	DENTSPLY® 454305
INTERNAL HEX USA (Zimmer Screw Vent™ MIS® Implants)	NP/RP/WP	19.012	30	Zimmer® URS2
	NP/RP/WP	19.012L	30	-
	NP/RP/WP	19.042	30	Zimmer® MHLAS / MIS® MN-S0160
	NP/RP/WP	19.312	30	-
	NP (MIS®)	19.327	35	-
	NP on implant for Multi-unit®	19.519	20	-
	RP on Multi-unit® abutment Ø4.8	19.110	15	-
EXT HEX USA (3i® Osseotite®)	RP on Multi-unit® abutment Ø4.8	19.300	15	-
	NP/RP/WP	19.034	20	BIOMET 3i® UNIHT
	RP on implant for Multi-unit®	19.512	20	-
INTERNAL HEX CLICK (3i® Certain®)	Low Profile® Abutment Ø4.8	19.106	10	BIOMET 3i® LPCTSH
	NP/RP/WP	19.011	20	BIOMET 3i® IUNIHT
	NP/RP/WP	19.311	20	-
EXTERNAL HEX BH (BioHorizons®)	Low Profile® Abutment Ø4.8	19.106	10	BIOMET 3i® LPCTSH
	NP	19.013	30	BioHorizons® 130-050
	NP	19.312	30	-
MULTI-UNIT®	RP/WP	19.014	30	BioHorizons® 140-050
	RP/WP	19.328	35	-
	Multi-unit® RP abutment Ø4.8	19.006	15	Nobel Biocare® 29287
ANKYLOS® C/X	Low Profile® Abutment Ø4.8	19.106	10	BIOMET 3i® LPCTSH
	Multi-unit® RP abutment Ø4.8	19.206	15	Nobel Biocare® 29285
	Multi-unit® RP abutment Ø4.8	19.306	15	-
	Multi-unit® RP abutment Ø4.8	19.110	15	-
	MIS® Multi-unit® RP abutment Ø4.8	19.300	15	-
	Multi-unit® WP abutment Ø6.0	19.031	15	Nobel Biocare® 31163
	Multi-unit® WP abutment Ø6.0	19.231	15	Nobel Biocare® 29286
	Multi-unit® WP abutment Ø6.0	19.331	15	-
	All platforms	19.048	20	-
	All platforms	19.050	20	-
	Universal 3.6	19.038	32	Dyna Dental 31A51
SPECIAL DESIGN (Dyna® System)	Octa	19.039	30	Dyna Dental 81A50
	Extension Level Octa	19.040	30	Dyna Dental 82E50

All DESS® products are manufactured according to ISO 9001:2008 and ISO 13485:2012 and bare the CE marking. 3i® and 3i® Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA. ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XIVE® and FRIADENT® and UniAbutment® are registered trademarks of DENTSPLY Implants. BioHorizons® registered trademark of BioHorizons, INC. Brånemark System®, NobelReplace® and Replace Select™ and NobelActive™ and NobelReplace® CC and NobelSpeed™ and Multi-unit® registered trademarks of Nobel Biocare AB, Sweden. IT™, Straumann® and synOcta® registered trademarks of Straumann Holding AG, Switzerland. MIS® registered trademark of MIS Implant Technologies Ltd. Zimmer Screw-Vent™ and SwissPlus® registered trademarks of Zimmer Dental INC, USA. Dyna® registered trademark of Dyna Dental Engineering BV. Locator® registered trademark of Zest Anchors INC. The type, reference and connection of each element should be checked. Some DESS® items may not be available in your country. These instructions replace all previous editions. For any detailed information about DESS® products please contact your local distributor.